

VS 300 S

The current version of the installation and operating instructions is available in the Download Center:



<http://qr.duerdental.com/7119100007>



Ambient temperature



Upper moisture limit




Only qualified specialists or employees trained by Dürr Dental are permitted to install, connect and start using the unit.




Assembly


1

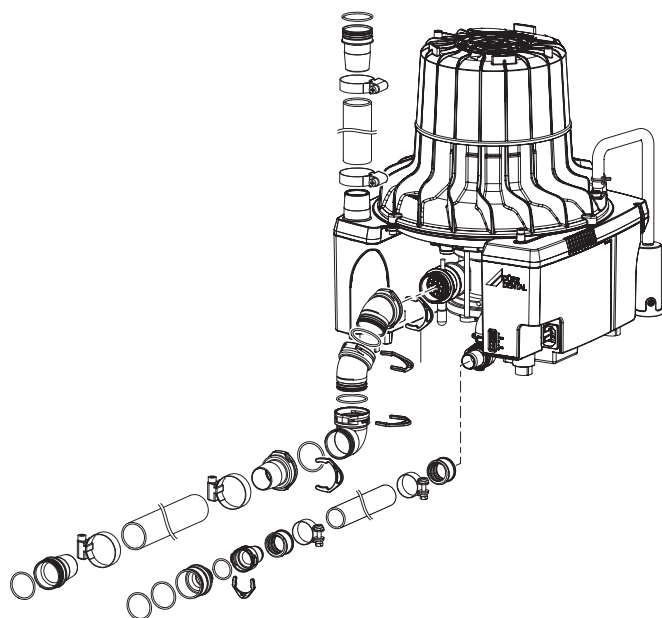

2

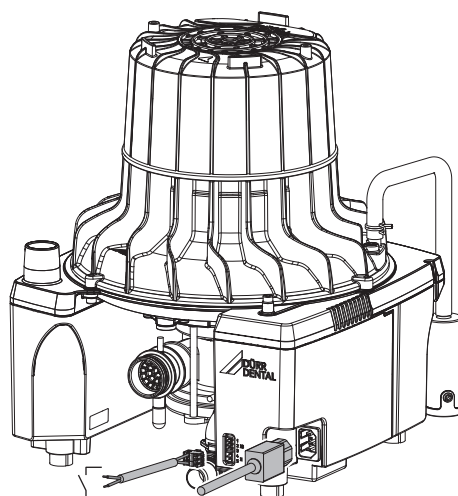


3

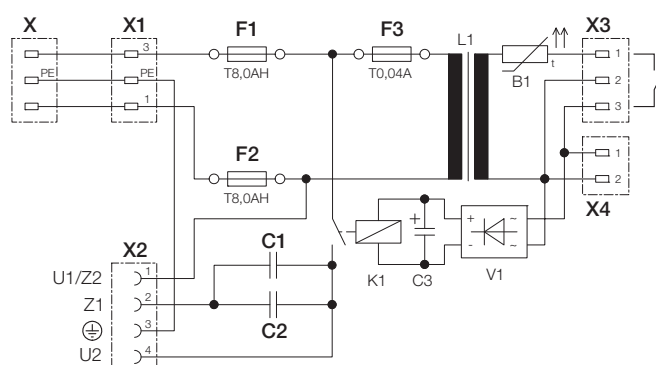

Usage

Disinfection and cleaning

After each patient: suction 0.5 l of water

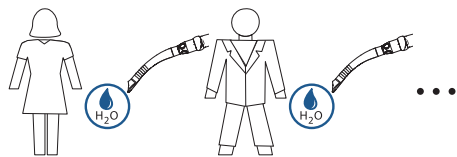

Daily: disinfection

Daily in the evening after finishing treatment; twice daily with high loads.



Always use:

- Orotol plus / Orotol plus pH 7
- Unit care system, e.g. OroCup



Wear protective gloves.



Wear protective goggles.

1




2



3



4

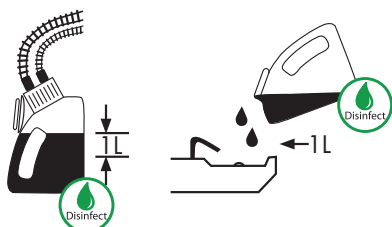

5



Cleaning: 2 x or 5 x per week

Min. 2 x per week, contact time 30 - 120 minutes

Alternatively: 5 x per week, contact time 5 - 15 minutes



Always use:

- MD 555 cleaner / MD 555 cleaner organic
- Unit care system, e.g. OroCup



Wear protective gloves.




Wear protective goggles.

1




2

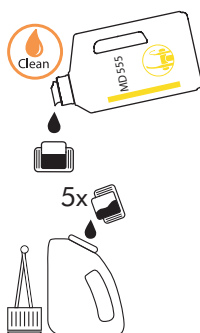


3



4

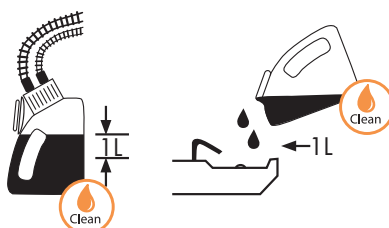

5

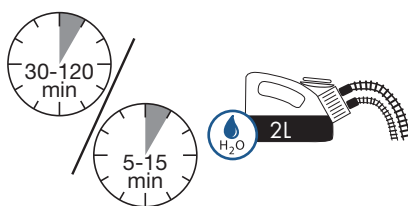

Cleaning the protective strainer


1

1.	2.	3.	4.	1.
----	----	----	----	----

Woche / week



1.

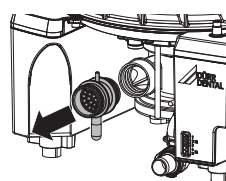

2.

