

# VS 600, VS 900 S, VS 1200 S

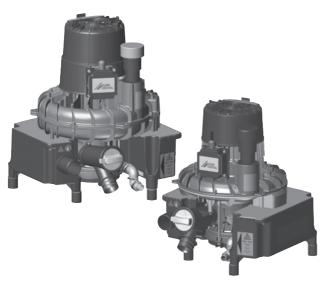

EN Installation and operating instructions



## **Contents**

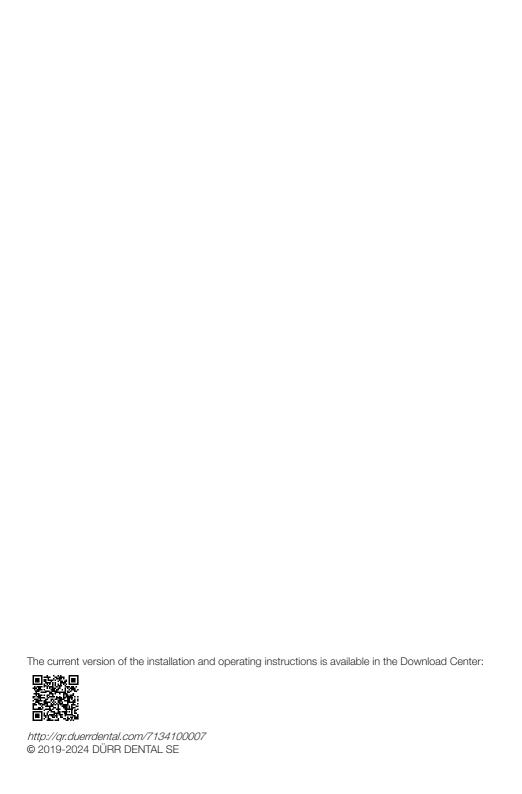

ln	nporta	ant information	
1	Abou	t this document	3
	1.1	Warnings and symbols	3
	1.2	Copyright information	4
2	Safet	y	4
	2.1	Intended purpose	4
	2.2	Intended use	4
	2.3	Improper use	4
	2.4	General safety information	5
	2.5	Combining devices safely	5
	2.6	Specialist personnel	5
	2.7	Notification requirement of seri-	
		ous incidents	5
	2.8	Electrical safety	5
	2.9	Only use original parts	5
	2.10	Transport	6
	2.11	Disposal	6
Pı	roduc	t description	
3	Over	/iew	7
	3.1	Scope of delivery	8
	3.2	Optional items	8
	3.3	Consumables	9
	3.4	Wear parts and replacement	
		parts	9
4		nical data	10
	4.1	VS 600	10
	4.2	VS 600	12
	4.3	VS 600	14
	4.4	VS 900 S	16
	4.5	VS 900 S	18
	4.6	VS 1200 S	20
	4.7	VS 1200 S	22
	4.8	Type plate	24
	4.9	Evaluation of conformity	24
5	Funct	tion	25



## Assembly

6	Requi	rements	27
	6.1	Installation/setup room	27
	6.2	Setup options	27
	6.3	Pipe materials	27
	6.4	Hose materials	27
	6.5	Information about electrical connections	27
	6.6	Information about connecting	
		cables	27
7	Syste	m components	28
	7.1	Control box	28
	7.2	Rinsing unit	28
	7.3	Flow accelerator	28
	7.4	Amalgam separator	28
	7.5	Surge tank	28
	7.6	Exhaust air filter	29
	7.7	Noise reduction	29
8	Install	ation	29
	8.1	Installation and routeing of hoses and pipes	29
	8.2	Electrical connections	31
	8.3	Testing suction power after installation	31
9	Comn	nissioning	32



## Usage

10	Disinf	ection and cleaning	33
	10.1	After every treatment	33
	10.2	Daily after the end of treatment .	33
	10.3	Once or twice a week before the	
		midday break	33
11	Clean	ing the protective strainer	34
	11.1	VS 600, VS 900 S, VS 1200 S .	34
12	Maint	enance	35



7134100007L02 2405V003

		4.5
Troub	acho	ntina
HOUD	COLIC	Othing

13	Tips for operators and service techni-	36
	cians	30
14	Transporting the unit	38



## Appendix

15	Handover record	39
16	Country representatives	40

## Important information

### About this document

These installation and operating instructions represent part of the unit.



The manufacturer and the distributor will not offer any guarantee or accept any liability for the safe operation and the safe functioning of the unit if the instructions and information in these installation and operating instructions are not complied with.

The German version of the installation and operating instructions is the original manual. All other languages are translations of the original manual. These installation and operating instructions apply to:

### VS 600

REF: 7128-01: 7128-01/002: 7128-01/021: 7128100900: 7128-02: 7128-02/002: 7128-02/003

### **VS 900 S**

REF: 7134-01; 7134-01/002; 7134-01/021; 7134100900; 7134-02; 7134-02/002; 7134-02/021

#### VS 1200 S

REF: 7138-02; 7138-02/002; 7138-02/021; 7138100900; 7138-02/003; 7138-03;

7138-03/002; 7138-03/003

#### 1.1 Warnings and symbols

#### Warnings

The warnings in this document are intended to draw your attention to possible injury to persons or damage to machinery.

The following warning symbols are used:



General warning symbol



Warning - dangerous high voltage



Warning - hot surfaces



Warning - automatic start-up of the unit



Biohazard warning

The warnings are structured as follows:

### SIGNAL WORD

### Description of the type and source of danger

Here you will find the possible consequences of ignoring the warning

> Follow these measures to avoid the danger.

The signal word differentiates between four levels of danger:

### DANGER

Immediate danger of severe injury or death

### WARNING

Possible danger of severe injury or death

#### CAUTION

Risk of minor injuries

#### NOTICE

Risk of extensive material/property damage

### Other symbols

These symbols are used in the document and on or in the unit:



Note, e.g. specific instructions regarding efficient and cost-effective use of the unit.



Refer to Operating Instructions.



Wear protective gloves.



Wear protective goggles.



Disconnect all power from the unit.



Refer to the accompanying electronic documents



Lower and upper temperature limits



Lower and upper humidity limits



Protective ground connection

7134100007L02 2405V003

## ⟨ ← xxxx CE labelling with the number of the notified body

SN Serial number

**REF** Order number

MD Medical device

HIBC Health Industry Bar Code (HIBC)

Manufacturer

### 1.2 Copyright information

All circuits, processes, names, software programs and units mentioned in this document are protected by copyright.

The Installation and Operating Instructions must not be copied or reprinted, neither in full nor in part, without written authorisation from the copyright owner.

## 2 Safety

The unit has been developed and designed in such a way that dangers are effectively ruled out if used in accordance with the Intended Use. Despite this, the following residual risks can remain:

- Personal injury due to incorrect use/misuse
- Personal injury due to mechanical effects
- Personal injury due to electrical shock
- Personal injury due to radiation
- Personal injury due to fire
- Personal injury due to thermal effects on skin
- Personal injury due to lack of hygiene, e.g. infection

### 2.1 Intended purpose

The suction unit provides the dental treatment unit with vacuum and volume flow.

### 2.2 Intended use

Working in combination with the suction unit with treatment unit, suction handpiece and cannula, the media used in dental treatment (e.g. water, saliva, dentine and amalgam) are removed by suction for disposal.

This unit is technically suitable for the aspiration of nitrous oxide (laughing gas). However, when assembling a system for aspiration of nitrous oxide, it is important to ensure that the other components in the system are also suitable for this purpose. Those responsible for setting up the system must assess this and approve and release the system for the aspiration of nitrous oxide.



Operation with nitrous oxide is only permitted if the exhaust air is transported from the unit to the outside of the building.

### 2.3 Improper use

Any use of this appliance / these appliances above and beyond that described in the Installation and Operating Instructions is deemed to be incorrect usage. The manufacturer cannot be held liable for any damage resulting from incorrect usage. The operator will be held liable and bears all risks.

- Do not use this device to aspirate flammable or explosive mixtures.
- Do not use the unit as a vacuum cleaner.
- Do not use chemicals containing chlorine or foaming chemicals.
- Operation in operating theatres of explosive areas is not permissible.
- The suction unit must not be set up in the immediate surroundings of the patient (minimum distance: 1.5 m).

#### 2.4 General safety information

- Always comply with the specifications of all guidelines, laws, and other rules and regulations applicable at the site of operation for the operation of this unit.
- Check the function and condition of the unit prior to every use.
- Do not convert or modify the unit.
- Comply with the specifications of the Installation and Operating Instructions.
- The Installation and Operating Instructions must be accessible to all operators of the unit at all times.

#### 2.5 Combining devices safely

Take care when connecting units together or to parts of other systems as there is always an element of risk (e.g. due to leakage currents).

- Only connect units when there can be no question of danger to operator or to patient.
- Only connect units when it is safe to do so and when there is no risk of damage or harm to the surroundings.
- If it is not completely clear from the data sheet of the unit that such connections can be safely made or if you are in any doubt, always get a suitably qualified person (e.g. the relevant manufacturer) to verify that the setup is safe.

Where applicable, the requirements for medical products have been taken into account in the development and construction of the device. As a result, this device is suitable for installation within medical supply equipment.

- Where this device is integrated in other medical supply equipment, the requirements of European Union Medical Device Regulation 2017/745 and the relevant standards must be observed.

#### 2.6 Specialist personnel

### Operation

Unit operating personnel must ensure safe and correct handling based on their training and knowledge.

 Instruct or have every operator instructed in handling the unit.

### The following groups are not permitted to operate or use a commercially operated unit:

- People without the necessary experience and knowledae
- People with reduced physical, sensory or mental capabilities
- Children

#### Installation and repairs

 Have the manufacturer or a qualified company authorised by the manufacturer perform mounting, new installations, modifications. expansions and repairs.

#### 2.7 Notification requirement of serious incidents

The operator/patient is required to report any serious incident that occurs in connection with the device to the manufacturer and to the competent authority of the Member State in which the operator and/or patient is established/resident.

#### 2.8 **Electrical safety**

- Comply with all the relevant electrical safety regulations when working on the unit.
- Never touch the patient and unshielded plug connections on the unit at the same time.
- Replace any damaged cables or plugs immediately.

#### 2.9 Only use original parts

- Only use accessories and optional articles named or authorised by the manufacturer.
- Only use only original wear parts and replacement parts.





The manufacturer and distributor accept no liability for damages or injury resulting from the use of non-approved accessories, optional accessories, or from the use of non-original wear parts or replacement parts.

The use of non-approved accessories, optional accessories or non-genuine wear parts / replacement parts (e.g. mains cables) can have a negative effect in terms of electrical safety and EMC.

### 2.10 Transport

The original packaging provides optimum protection for the unit during transportation.

If required, the original packaging for the unit can

If required, the original packaging for the unit can be ordered.



The manufacturer and the distributor do not accept liability, even during the warranty period, for damage during transportation due to improper packaging.

- Only transport the unit in its original packaging.
- Keep the packing materials out of the reach of children.

### 2.11 Disposal



The unit may be contaminated. Instruct the company disposing of the waste to take the relevant safety precautions.

- Decontaminate potentially contaminated parts before disposing of them.
- Uncontaminated parts (e.g. electronics, plastic and metal parts etc.) should be disposed of in accordance with the local waste disposal regulations.
- If you have any questions about the correct disposal of parts, please contact your dental trade supplier.




An overview of the waste keys for Dürr Dental products can be found in the download area:




http://gr.duerrdental.com/P007100155

## **Product description**

## Overview

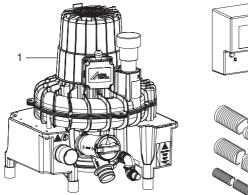

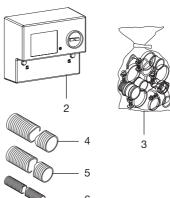

- 1 combination suction unit
- 2 Control box
- 3 Set of connection fittings
- 4 Suction hose
- 5 Exhaust air hose
- 6 Waste water hose LW 20



#### 3.1 Scope of delivery

The following items are included in the scope of delivery (possible variant-specific deviations due to country-specific requirements and/or import regulations): VS 600, 230 V, 1~, 50 Hz (basic unit) . . . . . . . . . . . . . . . . . 7128-01 VS 600, 230 V, 1~, 50 Hz (including accessories) . . . . . . . 7128-01/002 VS 600, 230 V, 1~, 50 Hz (Power Tower) . . . . . . . . . . . . . . . 7128-01/021 VS 600, 230 V, 1~, 50 Hz (including accessories) . . . . . . . 7128100900 VS 600, 230/400 V, 3~, 50/60 Hz (basic unit) . . . . . . . . . . . . . . . . . . 7128-02 VS 600, 400 V, 3~, 50/60 Hz (including accessories) . . . . . . . 7128-02/002 VS 600, 230 V, 3~, 50/60 Hz (including accessories) . . . . . . . . 7128-02/003 Control box 0700-500-50 for model 7128-01/002 0700-500-50 for model 7128100900 0732-100-56 for model 7128-02/002 0732-100-57 for model 7128-02/003 Set of connection fittings - Suction hose LW 50 - Exhaust air hose LW 50 (aluminium) Waste water hose LW 20 - OroCup VS 900 S. 230 V. 1~. 50 Hz (basic unit) . . . . . . . . . . . . . . . . . 7134-01 VS 900 S, 230 V, 1~, 50 Hz (including accessories) . . . . . . . . 7134-01/002 VS 900 S, 230 V, 1~, 50 Hz (Power Tower) . . . . . . . . . . . . . . . . 7134-01/021 VS 900 S, 230 V, 1~, 50 Hz (including accessories) . . . . . . . 7134100900 VS 900 S, 230/400 V, 3~, 50 Hz (basic unit) . . . . . . . . . . . . . . . . . . 7134-02 VS 900 S, 400 V, 3~, 50 Hz (including accessories) . . . . . . . . 7134-02/002

VS 900 S, 400 V, 3~, 50 Hz

- Control box 0732-100-55 for model 7134-01/002 0732-100-55 for model 7134100900 0732-100-56 for model 7134-02/002 - Set of connection fittings Suction hose LW 50 Exhaust air hose LW 50 (aluminium) Waste water hose LW 20 OroCup VS 1200 S. 230/400 V. 3~. 50 Hz (basic unit) . . . . . . . . . . . . . . . . . . 7138-02 VS 1200 S, 400 V, 3~, 50 Hz (including accessories) . . . . . . . . 7138-02/002 VS 1200 S, 400 V, 3~, 50 Hz (Power Tower) . . . . . . . . . . . . . . . . . 7138-02/021 VS 1200 S, 400 V, 3~, 50 Hz (including accessories) . . . . . . . . . 7138100900 VS 1200 S. 230 V. 3~. 50 Hz (including accessories) . . . . . . . . . 7138-02/003 VS 1200 S, 230/400 V, 3~, 60 Hz (basic unit) . . . . . . . . . . . . . . . . . . 7138-03 VS 1200 S. 400 V. 3~. 60 Hz (including accessories) . . . . . . . . 7138-03/002 VS 1200 S, 230 V, 3~, 60 Hz (including accessories) . . . . . . . . 7138-03/003 Control box 0732-100-61 for model 7138-02/002 0732-100-61 for model 7138100900 0732-100-59 for model 7138-02/003 0732-100-61 for Modell 7138-03/002 0732-100-59 for model 7138-03/003 Set of connection fittings - Suction hose LW 50

- Exhaust air hose LW 50 (aluminium)
- Waste water hose LW 20
- OroCup

#### Optional items 3.2

the
OO-C
1-00
5-50
1-00
1-51
1-50
1-00
0-50

Ventilation kit for cabinet installa-

Exhaust air valve . . . . . . . . . . . . 0732-020-00 Bacteria filter with housing \* . . . . 7120100000

\* Only required for V/VS 600, 2x

#### 3.3 Consumables

The following materials are consumed during operation of the device and must be ordered separately:

Orotol plus (2.5 litre bottle) . . . . CDS110P6150 MD 555 cleaner (2.5 litre bottle) . CCS555C6150

#### 3.4 Wear parts and replacement parts

The following working parts need to be changed at regular intervals (refer to the "Maintenance" section):

Nonreturn valve (pack of 3) . . . . 7128-100-03E



Information about replacement parts is available from the portal for authorised specialist dealers at:

www.duerrdental.net

## 4 Technical data

### 4.1 VS 600

Electrical data		7128-01 7128100900
Rated voltage	V	230, 1~
Mains frequency	Hz	50
Active power	kW	1
Nominal current	А	≤ 5.0
Start-up current	А	~ 22
Motor protection *	А	-
Type of protection		IP 24
Protection class		l

### \* Motor winding overheat protector

Connections		
Suction connection (outside)	mm	Ø 50
Exhaust air connection (external)	mm	Ø 50
Drain connection, DürrConnect	mm	Ø 20
Media		
Max. rate of fluid flow	l/min	10
Max. suction height	cm	80

Classification in accordance with EN ISO 10637	
Classification based on separation of sol-	
ids and liquids	Semi-dry suction system
Classification based on flow rate	Type 1

General data			
Max. number of users		2	
Duty cycle	%	100 (S1)	
Heat generation rate	MJ/h	3.6	
Dimensions (H x W x D)	cm	48 x 41 x 39	
Weight, approx.	kg	22	
Sound pressure level * approx. without housing with housing	dB(A) dB(A)	65 57	

<sup>\*</sup> Sound pressure level in accordance with ISO 3746

Ambient conditions during storage and transport			
Temperature	°C	-10 - +60	

### Ambient conditions during storage and transport


Relative humidity < 95

#### Ambient conditions during operation °C Temperature +10 - +40 % < 70 Relative humidity

### Classification

Medical Device Class (MDR) lla

### Characteristic curve according to EN ISO 10637

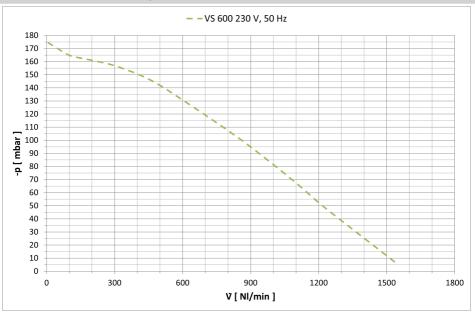

Characteristic curve of: 7128-01, 7128100900



### 4.2 VS 600

Electrical data		712	8-02
Rated voltage	V	230	), 3~
Mains frequency	Hz	50	60
Active power	kW	1.0	1.4
Nominal current	А	≤ 3.8	≤ 4.0
Start-up current	А	~ 16	~ 14
Motor protection *	А	4	.5
Type of protection		IP	24
Protection class			l

### \* Max. permitted setting

Connections		
Suction connection (outside)	mm	Ø 50
Exhaust air connection (external)	mm	Ø 50
Drain connection, DürrConnect	mm	Ø 20

Media		
Max. rate of fluid flow	l/min	10
Max. suction height	cm	80

Classification in accordance with EN ISO 10637	
Classification based on separation of solids and liquids	Semi-dry suction system
Classification based on flow rate	Type 1

General data			
Max. number of users		2	2
Duty cycle	%	100	(S1)
Heat generation rate	MJ/h	3.6	5
Dimensions (H x W x D)	cm	48 x 4	1 x 39
Weight, approx.	kg	2	2
Sound pressure level * approx. without housing with housing	dB(A) dB(A)	65 57	68 60

<sup>\*</sup> Sound pressure level in accordance with ISO 3746

Ambient conditions during storage and transport			
Temperature	°C	-10 - +60	
Relative humidity	%	< 95	

12 | EN 7134100007L02 2405V003


Ambient conditions during operation	n	
Temperature	°C	+10 - +40
Relative humidity	%	< 70

### Classification

Medical Device Class (MDR)

lla

### Characteristic curve according to EN ISO 10637

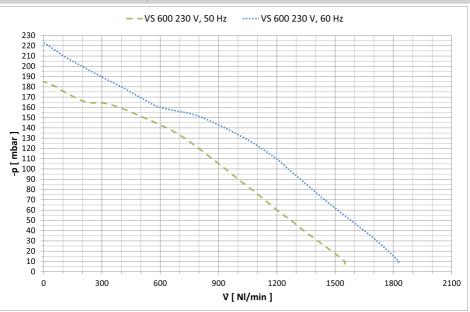

Characteristic curve of: 7128-02



### 4.3 VS 600

Electrical data		712	8-02
Rated voltage	V	400	), 3~
Mains frequency	Hz	50	60
Active power	kW	1.0	1.4
Nominal current	А	≤ 2.2	≤ 2.3
Start-up current	А	~ 16	~ 14
Motor protection *	Α	2	.5
Type of protection		IP	24
Protection class			l

### \* Max. permitted setting

Connections		
Suction connection (outside)	mm	Ø 50
Exhaust air connection (external)	mm	Ø 50
Drain connection, DürrConnect	mm	Ø 20

Media		
Max. rate of fluid flow	l/min	10
Max. suction height	cm	80

Classification in accordance with EN ISO 10637	
Classification based on separation of solids and liquids	Semi-dry suction system
Classification based on flow rate	Type 1

General data			
Max. number of users		2	2
Duty cycle	%	100	(S1)
Heat generation rate	MJ/h	3.6	5
Dimensions (H x W x D)	cm	48 x 4	1 x 39
Weight, approx.	kg	2	2
Sound pressure level * approx. without housing with housing	dB(A) dB(A)	65 57	68 60

<sup>\*</sup> Sound pressure level in accordance with ISO 3746

Ambient conditions during storage and transport			
Temperature	°C	-10 - +60	
Relative humidity	%	< 95	

14 | EN 7134100007L02 2405V003


Ambient conditions during operat	ion	
Temperature	°C	+10 - +40
Relative humidity	%	< 70

### Classification

Medical Device Class (MDR)

lla

### Characteristic curve according to EN ISO 10637

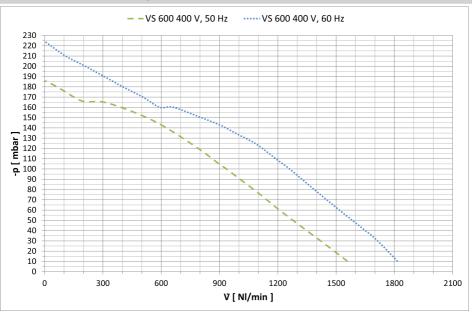

Characteristic curve of: 7128-02



### 4.4 VS 900 S

Electrical data		7134-01 7134100900
Rated voltage	V	230, 1~
Mains frequency	Hz	50
Active power	kW	1.6
Nominal current	А	≤ 7.4
Start-up current	А	~ 32
Motor protection *	А	10.0
Type of protection		IP 24
Protection class		I

### \* Max. permitted setting

Connections		
Suction connection (outside)	mm	Ø 50
Exhaust air connection (external)	mm	Ø 50
Drain connection, DürrConnect	mm	Ø 20

Media		
Max. rate of fluid flow	l/min	16
Max. suction height	cm	80

Classification in accordance with EN ISO 10637	
Classification based on separation of solids and liquids	Semi-dry suction system
Classification based on flow rate	Type 1

General data			
Max. number of users		3	
Duty cycle	%	100 (S1)	
Heat generation rate	MJ/h	5.8	
Dimensions (H x W x D)	cm	52 x 41 x 43	
Weight, approx.	kg	29	
Sound pressure level * approx. without housing with housing	dB(A) dB(A)	65 61	

<sup>\*</sup> Sound pressure level in accordance with ISO 3746

Ambient conditions during storage and transport			
Temperature	°C	-10 - +60	
Relative humidity	%	< 95	

16 | EN 7134100007L02 2405V003

Ambient conditions during opera	tion	
Temperature	°C	+10 - +40
Relative humidity	%	< 70

### Classification

Medical Device Class (MDR)

Characteristic curve according to EN ISO 10637

lla

### - - VS 900 S 230 V, 50 Hz **aba** 100 90 80

Ϋ [ NI/min ]

Characteristic curve of: 7134-01, 7134100900



### 4.5 VS 900 S

Electrical data		7134	l-02	
Rated voltage	V	230, 3~	400, 3~	
Mains frequency	Hz	50	)	
Active power	kW	1.7	1.7	
Nominal current	А	≤ 6.3	≤ 3.6	
Start-up current	А	~ 42	~ 25	
Motor protection *	А	6.3	4.0	
Type of protection		IP 24		
Protection class		I		

### \* Max. permitted setting

Connections		
Suction connection (outside)	mm	Ø 50
Exhaust air connection (external)	mm	Ø 50
Drain connection, DürrConnect	mm	Ø 20

Media		
Max. rate of fluid flow	l/min	16
Max. suction height	cm	80

Classification in accordance with EN ISO 10637	
Classification based on separation of solids and liquids	Semi-dry suction system
Classification based on flow rate	Type 1

General data			
Max. number of users		3	
Duty cycle	%	100 (S1)	
Heat generation rate	MJ/h	6.1	
Dimensions (H x W x D)	cm	52 x 41 x 43	
Weight, approx.	kg	28	
Sound pressure level * approx. without housing with housing	dB(A) dB(A)	65 61	

<sup>\*</sup> Sound pressure level in accordance with ISO 3746

Ambient conditions during storage	and transport	
Temperature	°C	-10 - +60
Relative humidity	%	< 95

18 | EN 7134100007L02 2405V003


Ambient conditions during operation		
Temperature	°C	+10 - +40
Relative humidity	%	< 70

### Classification

Medical Device Class (MDR)

lla

### Characteristic curve according to EN ISO 10637

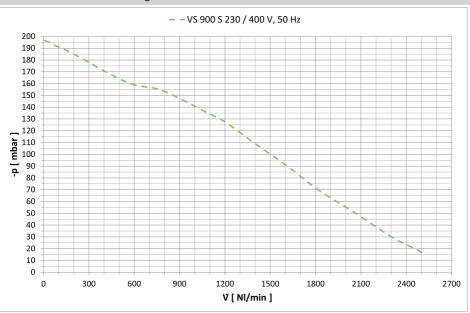

Characteristic curve of: 7134-02



### 4.6 VS 1200 S

Electrical data	7138-02 7138100900			
Rated voltage	V	230, 3~	400, 3~	
Mains frequency	Hz	50		
Active power	kW	1.8	1.8	
Nominal current	А	≤ 6.2	≤ 3.6	
Start-up current	Α	~ 43	~ 25	
Motor protection *	А	6.3	4.0	
Type of protection		IP 24		
Protection class		l		

### \* Max. permitted setting

Connections		
Suction connection (outside)	mm	Ø 50
Exhaust air connection (external)	mm	Ø 50
Drain connection, DürrConnect	mm	Ø 20

Media		
Max. rate of fluid flow	l/min	24
Max. suction height	cm	80

Classification in accordance with EN ISO 10637	
Classification based on separation of solids and liquids	Semi-dry suction system
Classification based on flow rate	Type 1

	4
%	100 (S1)
MJ/h	6.5
cm	52 x 41 x 43
kg	28
dB(A) dB(A)	65 61
	MJ/h cm kg dB(A)

<sup>\*</sup> Sound pressure level in accordance with ISO 3746

Ambient conditions during storage and transport			
Temperature °C -10 - +60			
Relative humidity	%	< 95	

20 | EN 7134100007L02 2405V003


Ambient conditions during operation	on	
Temperature	°C	+10 - +40
Relative humidity	%	< 70

### Classification

Medical Device Class (MDR)

lla

### Characteristic curve according to EN ISO 10637

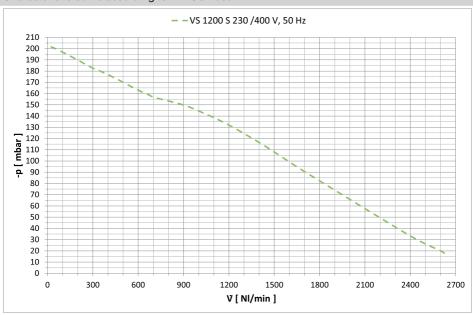

Characteristic curve of: 7138-02, 7138100900



### 4.7 VS 1200 S

Electrical data		7138	3-03
Rated voltage	V	230, 3~	400, 3~
Mains frequency	Hz	6	0
Active power	kW	2.3	2.3
Nominal current	Α	≤ 6.8	≤ 3.9
Start-up current	А	~ 30	~ 17
Motor protection *	А	7	4.0
Type of protection	IP 24		
Protection class			

### \* Max. permitted setting

Connections		
Suction connection (outside)	mm	Ø 50
Exhaust air connection (external)	mm	Ø 50
Drain connection, DürrConnect	mm	Ø 20

Media		
Max. rate of fluid flow	I/min	24
Max. suction height	cm	80

Classification in accordance with EN ISO 10637	
Classification based on separation of solids and liquids	Semi-dry suction system
Classification based on flow rate	Type 1

General data		
Max. number of users		4
Duty cycle	%	100 (S1)
Heat generation rate	MJ/h	8.3
Dimensions (H x W x D)	cm	52 x 41 x 43
Weight, approx.	kg	28
Sound pressure level * approx. without housing with housing	dB(A) dB(A)	69 65

<sup>\*</sup> Sound pressure level in accordance with ISO 3746

Ambient conditions during storage and transport		
Temperature	°C	-10 - +60
Relative humidity	%	< 95

22 | EN 7134100007L02 2405V003


Ambient conditions during operation			
Temperature	°C	+10 - +40	
Relative humidity	%	< 70	

### Classification

Medical Device Class (MDR)

lla

### Characteristic curve according to EN ISO 10637

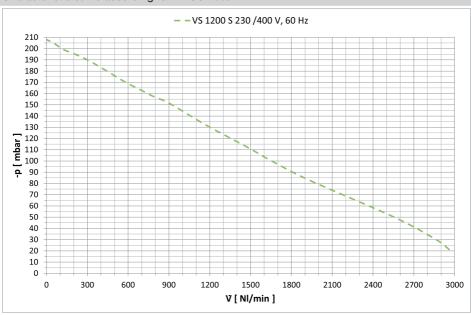


Characteristic curve of: 7138-03

### 4.8 Type plate

### **VS 600**


The type plate is located on the upper part of the turbine housing.

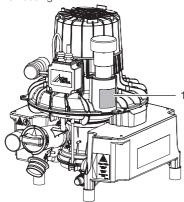

1 Type plate

### VS 900 S, VS 1200 S

The type plate is located on the upper part of the turbine housing.

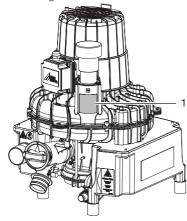


1 Type plate

## 4.9 Evaluation of conformity

This device has been subjected to conformity acceptance testing in accordance with the current relevant European Union guidelines. This equipment conforms to all relevant requirements.

24 | EN 7134100007L02 2405V003

#### **Function** 5

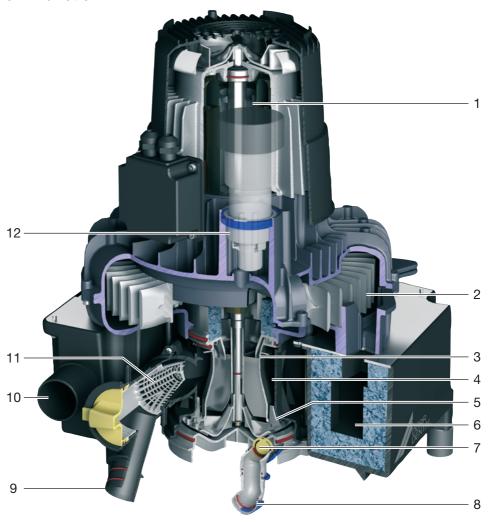

- Motor 1
- 2 Turbine wheel
- 3 Separation turbine
- Cyclone separator 4
- 5 Waste water pump
- 6 Exhaust air muffler
- 7 Diaphragm valve
- 8 Waste water connection
- 9 Intake nozzle
- 10 Exhaust air connection



- 11 Protective strainer
- 12 Auxiliary air valve

The VS suction units are used in wet suction systems. The suction units can be installed on the same floor as the treatment units or on the floor underneath. The necessary air flow and vacuum are generated by a rapidly rotating impeller. The impeller, separation turbine and waste water pump are driven by

An auxiliary air valve on the turbine housing protects the suction unit against overheating and provides uniform suction power. When an appropriate vacuum for the machine is applied, approx. 300 l/min of air is aspirated through the suction cannula.



The auxiliary air valve is matched to the suction machine. It must not be modified or replaced with an auxiliary air valve with a different setting.

The aspirated mixture of liquids, solid particles and air passes through the inlet socket into the suction unit. The protective strainer retains coarse solid particles.

Inside the separation unit, the aspirated liquids and solid particles are separated from the suction air through a two-stage separation system. This separation system consists of a cyclone separator and a separation turbine. The suction process runs continuously.

The aspirated mixture flows into the cyclonic separator, where it is set into a spiral motion. In this first stage, the resulting centrifugal forces force liquid constituents and remaining solid particles against the outside wall of the separation chamber of the cyclone separator. Initially, this only effects coarse separation of the liquid. In the subsequent second stage, the separation turbine effects a fine separation, in which the remaining liquid is separated from the air flow that has carried it to this point.

The waste water pump transports the liquid from the centrifuge and the fine solid particles it contains through the waste water connection into the central waste water network. A diaphragm valve located in the waste water connection prevents liquid from the drain being suctioned back in.

The exhaust air from the suction unit should be guided out of the building, via the roof where possible. We recommend incorporating a bacterial filter in the exhaust air line. In addition, it is possible to install a noise-reducing muffler in the exhaust air line in order to reduce the noise generated by the unit and by the air flow.



### Requirements

The unit can be installed on the same level as the surgery room or in a floor below.



Further information can be found in our suction planning information leaflet. Order number9000-617-03/...

#### Installation/setup room 6.1

The room chosen for set up must fulfil the followina requirements:

- Closed, dry, well-ventilated room
- Should not be a room made for another purpose (e. g. boiler room or wet cell)
- When installing in a cabinet the inlet and outlet ventilation slots must be present: minimum free cross-section at least 120 cm<sup>2</sup>.
- Forced ventilation (fan) must be provided if there is a risk that the recommended room air temperature could be exceeded. The air flow performance must be at least 2 m<sup>3</sup>/min.
- Do not cover cooling slots or openings with housing installations; ensure sufficient clearance to the openings to permit sufficient cooling.

#### 6.2 Setup options

The following options for setting up the unit are available:

- Wall installation using a Dürr Dental wall mounting
- In a ventilated cabinet
- In a Dürr Dental noise reducing housing

#### Pipe materials 6.3

Only use vacuum-sealed HT-waste pipes manufactured from the following materials:

- Polypropylene (PP),
- Chlorinated polyvinyl chloride (PVC-C),
- Plasticizer-free polyvinyl chloride (PVC-U),
- Polyethylene (PE).

#### The following materials must not be used:

- Acrylonitrile-butadiene-styrene (ABS),
- Styrene copolymer blends (e.g. SAN + PVC).

#### 64 Hose materials

For waste connections and suction lines only use the following hose types:

- Flexible spiral hoses made of PVC with integrated spiral or equivalent hoses
- Hoses that are resistant to dental disinfectants. and chemicals



Plastic hoses will display signs of ageing over time. Therefore, they should be inspected regularly and replaced as necessarv.

### The following types of hoses must not be used:

- Rubber hoses
- Hoses made completely of PVC
- Hoses that are not sufficiently flexible

#### Information about electrical 6.5 connections

- Ensure that the electrical connections to the mains power supply are established in accordance with current valid national and local regulations and standards governing the installation of low voltage units in medical facilities.
- Install an all-pole disconnect switch with a contact opening width of at least 3 mm in the electrical connection to the mains power sup-
- Observe the current consumption of the devices that are to be connected.

### Electrical fusing

LS switch 16 A, characteristic B, C and D in accordance with 60898.

#### 6.6 Information about connecting cables

The diameter of the connections depends on the current consumption, length of line and the ambient temperature of the unit. Information concerning the current consumption can be found in the Technical Data supplied with the particular unit to be connected.

The following table lists the minimum diameters of the connections in relation to the current consumption:

Current consumption of unit [A]	Cross-section [mm <sup>2</sup> ]
> 10 and < 16	1.5
> 16 and < 25	2.5

### Assembly

Current consumption of unit [A]	Cross-section [mm <sup>2</sup> ]
> 25 and < 32	4
> 32 and < 40	6
> 40 and < 50	10
> 50 and < 63	16

### Mains supply cable

Installation type	Line layout (minimum requirements)
Fixed installation	<ul> <li>Plastic sheathed cable (e.g. type NYM-J)</li> </ul>
Flexible	<ul><li>PVC flexible line (e.g. H05 VV-F)</li></ul>
	or
	<ul> <li>Rubber connection (e.g. H05 RN-F or H05 RR-F)</li> </ul>

#### Control cable

24 V protective low voltage for:

- Hose manifold
- Place selection valve
- Spittoon valve

Installation type	Line layout (minimum requirements)
Fixed installation	<ul> <li>Shielded sheathed cable (e.g. (N)YM (St)-J)</li> </ul>
Flexible	PVC data cable with shielded cable sheathing, as used for telecommunications and IT processing systems (e.g. type LiYCY)  or  Lightweight PVC control cable with shielded
	cable sheathing



Connect the shielding of the cables in accordance with the regulations.

## System components

The system components listed below are required or recommended for various procedures or for installation.

#### 7.1 Control box

The unit is connected via a control box. The control box is either included in the scope of delivery or must be ordered separately. In some units, the control system is built in.

#### 7.2 Rinsing unit

It is recommended that the suction system is equipped with a rinsing unit, e.g. in the treatment unit. The rinsing unit provides a small amount of water during aspiration. This dilutes the aspirated fluids (blood, saliva, rinsing water etc.), which can then be transported more effectively.

#### 7.3 Flow accelerator

In order to keep the suction system free of deposits, a flow accelerator can be fitted in conjunction with a spittoon valve. When using a bowl rinse system, water will collect before the flow accelerator. The next time suction takes place using the large cannula, the collected fluid is transported in surges and at high speed to the suction system. This ensures automatic cleaning of the suction pipes.


#### 7.4 Amalgam separator

The amalgam separator is designed to separate out and trap the heavy metal particles and amalgam dust that the suction unit aspirates from drilled fillings. The amalgam separator is installed in the drain behind the separation unit of the suction unit. The amount of fluid coming from the suction unit must not exceed the maximum permitted quantity of fluid that can be handled by the amalgam separator. Depending on the installation and on national regulations, a second amalgam separator may need to be installed.

#### 7.5 Surge tank

If the suction unit is combined with an amalgam separator, this requires the installation of a surge tank. The surge tank reduces pressure peaks caused by the waste water pump of the suction unit and acts as a buffer against temporary rises in the volume of water.

The surge tank can also be used if the waste water is fed directly into the building waste water system. this case the waste water from the suction unit is diverted to the building drainage system under zero pressure.

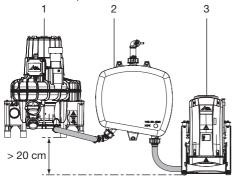

- 1 VS 900 S combination suction unit
- 2 Pressure equalisation tank
- 3 Amalgam separator CA 4

### 7.6 Exhaust air filter

For hygienic reasons, we recommend the installation of a bacteria filter in the exhaust air line. If the unit is installed in the surgery and the exhaust air cannot be discharged to the outdoors, it is essential to install a bacteria filter. Depending on the type and condition of the bacteria filter, it will need to be replaced every 1-2 years at the latest.



The separation integrated in the system does not retain bacteria; this is why we recommend installing a suitable filter in the exhaust air system.

### 7.7 Noise reduction

If the noise level from the exhaust air vent or the flow noise generated is too high, noise reduction can be installed in the exhaust air line.

### 8 Installation




The actual connection can vary depending on the chosen installation option. The connection shown is only an example.

## 8.1 Installation and routeing of hoses and pipes

- Establish connections between the pipe system and the unit using the flexible hoses supplied. This will prevent vibrations from being transmitted to the pipe system.
- The connection between the pipe line and unit suction connection should be kept as short as possible and straight, without bends.
- Install the drain hoses with a downward gradient so that the waste water can drain off.
- Waste water connections must be implemented in accordance with applicable local and national regulations.

### VS 600, VS 900 S, VS 1200 S

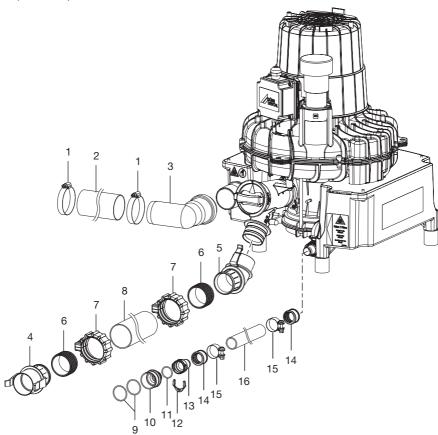

- 1 Hose clip 40-60 mm
- 2 Waste air pipe (aluminium)Ø 50 mm inside
- 3 Elbow DN 50 / 87°
- 4 Straight adaptor bush
- 5 Connecting sleeve
- 6 Sealing sleeve
- 7 Union nut
- 8 Suction hose Ø 50 mm
- 9 O-ring Ø 30x2
- 10 Connector Ø 36 mm (external)
- 11 O-ring Ø 20x2 mm
- 12 Securing ring
- 13 Hose sleeve Ø 20 mm
- 14 Hose sleeve
- 15 Hose clamp Ø 28 mm
- 16 Waste water hose Ø 20 mm (internal)

30 | EN

#### 8.2 **Electrical connections**

#### WARNING

Electric shock due to incorrectly connected device

> Never install a mains plug instead of the fixed connection.



#### NOTICE

### Short circuit due to defective lead

- Do not route wires near hot surfaces.
- Before connecting, check to make sure that the mains voltage matches the voltage specifications on the type plate.
- Only connect the unit to an original control
- Connect the power supply line from the control box to the appropriate terminals in the motor terminal box.



- Insert screwdriver into terminal strip to open spring clamp.
- Insert stranded wire into spring clamp.
- Remove screwdriver.

### 1/N/PE AC 230V

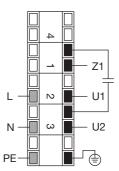


Fig. 1: Terminal assignment for 1-phase units

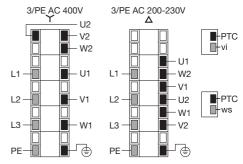


Fig. 2: Terminal assignment for 3-phase units

#### Testing suction power after 8.3 installation

The following procedure is recommended for measuring the maximum negative pressure of the suction system for comparison with the performance curve:

- 1. Lift a suction hose off a treatment unit and connect a pressure gauge (relative pressure). The connection adapter to the pressure gauge must be leak tight.
- 2. Read the displayed negative pressure and compare it against the performance curve.

#### Assembly



## Commissioning



In many countries technical medical products and electrical devices are subject to regular checks at set intervals. The owner must be instructed accordingly.



#### NOTICE

Interference caused by larger particles such as pieces of tooth or fillings

- > Do not operate the unit without a coarse filter
- 1. Check that the coarse filters are installed in the suction system (e.g. in the spittoon).
- 2. Turn on the unit power switch or the main surgery switch.
- Carry out a function check of the device.
- Check all connections for leak tightness.
- Carry out an electrical safety check in accordance with applicable regulations (e.g. regulations concerning set up, operation and application of medical devices) and record the results as appropriate, e.g. in the technical log book.
- Carry out and document the instruction and handover for the unit.



A sample handover report is included in the attachment.

## Usage

### 10 Disinfection and cleaning



### NOTICE

### Device malfunctions or damage due to use of incorrect media

Guarantee claims may become invalid as a result.

- > Do not use any foaming agents such as household cleaning agents or instrument disinfectants.
- > Do not use abrasive cleaners.
- > Do not use agents containing chlorine.
- > Do not use any solvents like acetone.

As a general rule use:

- for disinfection and cleaning: Orotol plus or Orotol ultra
- for cleaning: MD 555 cleaner

Only these products have been tested by Dürr Dental.

When using prophy powders, water-soluble Lunos Prophy Powders are recommended in order to protect the suction systems (Dürr Dental).

#### 10.1 After every treatment

1. Aspirate a glass of cold water through the large and the small suction hoses. Do this even if only the small suction hose was actually used during treatment.





Suction through the large suction hose causes a large amount of air to be drawn up, thereby considerably increasing the cleaning effect.

#### 10.2 Daily after the end of treatment



After higher workloads before the midday break and in the evening

The following are required for disinfection/cleaning:

- ✓ Non-foaming disinfectant/cleaning agent that is compatible with the materials.
- ✓ Unit care system, e.g. OroCup
- 1. To pre-clean, suck up 2 litres of water with the care system.
- 2. Aspirate the disinfection/cleaning agent with the care system.

### Once or twice a week before 10.3 the midday break



Under harsher conditions (e.g. hard water or frequent use of prophy powders) 1x daily before the midday break

The following are required for cleaning:

- ✓ Special non-foaming detergent for suction units that is compatible with the materials.
- ✓ Unit care system, e.g. OroCup
- To pre-clean, suck up 2 litres of water with the care system.
- Aspirate the cleaning agent with the care system.
- 3. Rinse with ca. 2 I water after the application time.

## 11 Cleaning the protective strainer




### WARNING

### Infection due to contaminated unit

- Clean and disinfect the suction before working on the unit.
- > Wear protective equipment when working (e. g. impermeable gloves, protective goggles and mouth and nose protection).

### 11.1 VS 600, VS 900 S, VS 1200 S

- Loosen the protective strainer by twisting it half a turn anti-clockwise.
- Pull out the protective strainer from the connection piece on the separation housing.

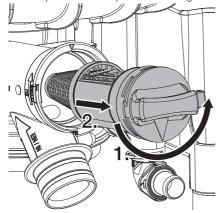

- 3. Clean the protective strainer.
- Push the protective strainer with the opening facing upwards back into the connection piece on the separation housing.
- Screw in the protective strainer clockwise as far as it will go.

### 12 Maintenance



All maintenance work must be performed by a qualified expert or by one of our Service Technicians.



### WARNING

### Infection due to contaminated unit

- > Clean and disinfect the suction before working on the unit.
- > Wear protective equipment when working (e. g. impermeable gloves, protective goggles and mouth and nose protection).



Prior to working on the unit or in case of danger, disconnect it from the mains.

Maintenance interval	Maintenance work
Every three months	> Check the filter at the device suction connection and clean if necessary.
Annually	> Check the waste valve for correct operation and replace it if necessary. *
Every 1-2 years	Replace the exhaust air filter (where fitted). *
Every 2 years	Check the auxiliary air valve for correct operation and clean/replace it as required. *

Only to be performed by service technicians.



## ? Troubleshooting

## 13 Tips for operators and service technicians



Any repairs exceeding routine maintenance may only be carried out by qualified personnel or our service.



### WARNING

#### Infection due to contaminated unit

- > Clean and disinfect the suction before working on the unit.
- > Wear protective equipment when working (e. g. impermeable gloves, protective goggles and mouth and nose protection).



Prior to working on the unit or in case of danger, disconnect it from the mains.

Error	Possible cause	Remedy
Device does not start	No mains voltage	<ul> <li>Check the mains supply voltage. *</li> <li>Check the fuses and replace if necessary. *</li> </ul>
	Undervoltage	Measure the supply voltage; call an electrician if necessary.
	Motor protection switch set too low	Measure current, set the motor protection switch to the measured value plus safety margin. *
	Motor protection switch defective	Check the motor protection switch; replace if defective. *
	Capacitor defective	Measure capacitance and replace if necessary. *
	Turbine is blocked by solid particles or sticky soiling	Disassemble the unit and clean the turbine. *
The unit generates unusual noises	Solid particles in the turbine chamber	<ul><li>Disassemble the unit and clean the turbine and housing.</li></ul>

Error	Possible cause	Remedy	
Water leaking from the exhaust air connection	Membrane valve blocked	Check the membrane valve at the waste water connection and if necessary clean or replace. *	
	Foam in turbine due to use of incorrect cleaning and disinfectant agents	Use non-foaming cleaning and disinfectant agents.	
	Build-up of condensate in the exhaust air line	Check the pipe system; avoid over-cooling. *	
	Waste water line/siphon trap blocked	Clean the waste water line/ siphon trap. *	
Suction performance too low	Coarse filter blocked	Clean the coarse filter at the intake connection.	
	Leak in the suction line	Check and if necessary establish leak-tightness of suction system and connec- tions. *	
	Mechanical sluggishness of turbine caused by soiling	Disassemble the unit and clean the turbine. *	

Only to be done by service technicians.



## 14 Transporting the unit



### WARNING

### Infection due to contaminated unit

- > Disinfect the unit before transport.
- > Close all media connections.



Wear protective equipment to avoid any risk of infection (e.g. liquid-tight protective gloves, protective goggles, face mask).

- Prior to disassembly, clean and disinfect the suction unit and the unit via aspiration of a suitable disinfectant approved by the manufacturer.
- Disinfect a defective unit using a suitable surface disinfection agent.
- 3. Seal all connections with sealing caps.
- Pack the unit securely in preparation for transport.

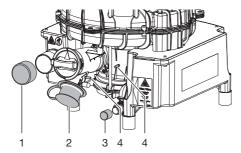


Fig. 3: VS 600, VS 900 S, VS 1200 S

- 1 Exhaust air connection sealing cap
- 2 Suction connection sealing caps
- 3 Water outflow sealing cap
- 4 Rinse connection sealing cap



### 15 Handover record

This document confirms that a qualified handover of the medical device has taken place and that appropriate instructions have been provided for it. This must be carried out by a qualified adviser for the medical device, who will instruct you in the proper handling and operation of the medical device.

Product name	Order number (	REF)	Serial number (SN)					
<ul> <li>□ Visual inspection of the packaging for any damage</li> <li>□ Unpacking the medical device and checking for damage</li> <li>□ Confirmation of the completeness of the delivery</li> <li>□ Instruction in the proper handling and operation of the medical device based on the operating instructions</li> </ul> Notes:								
Name of person receiving instruction: Signature:								
Name and address of the qualified adviser for the medical device:								
Date of handover:		Signature of the qualified adviser for the medical device:						

## Country representatives

### Country

### GB



### Address

#### **UK Responsible Person:**

Duerr Dental (Products) UK Ltd. 14 Linnell Way Telford Way Industrial Estate Kettering, Northants NN 16 8PS

#### IJΑ



**UA.TR.099** 

### Уповноважений представник в Україні:

Приватне підприємство "Галіт" вул. 15 квітня, 6Є, с. Байківці, Тернопільський р-н,

47711, Україна тел.: 0800 502 998; +38 050 338 10 64 www.galit.te.ua; e-mail: office@galit.te.ua

Виробник: Дюрр Дентал ЕсЕ Хьопфігхаймер Штрассе 17, Д-74321 Бітігхайм-Біссінген, Німеччина

email: info@duerrdental.com

#### CN

备案人/生产企业: DÜRR DENTAL SE 德国迪珥齿科股份公司

住所/生产地址: Höpfigheimer Str. 17, 74321 Bietigheim-Bissingen, Ger-

many 联系方式:

电话: +497142705-0 邮箱: info@duerrdental.com 网址: www.duerrdental.com

代理人/售后服务单位:迪珥医疗器械(上海)有限公司 住所:上海市长宁区天山路 641 号 2 号楼 (20 幢) 303 室

联系方式:

电话: +86 21 6381 0270 传真: +86 21 6381 0290 邮箱:info@duerr.cn

网址: http://www.duerrdental.com



### Hersteller / Manufacturer:

DÜRR DENTAL SE Höpfigheimer Str. 17 74321 Bietigheim-Bissingen Germany

Fon: +49 7142 705-0 www.duerrdental.com info@duerrdental.com

