

## Tyscor VS 2 Plus / VS 4

### Central combination suction unit

The current version of the installation and operating instructions is available in the Download Center:



<http://qr.duerdental.com/7188100021>



Ambient temperature



Upper moisture limit




Only qualified specialists or employees trained by Dürr Dental are permitted to install, connect and start using the unit.




## Assembly


1

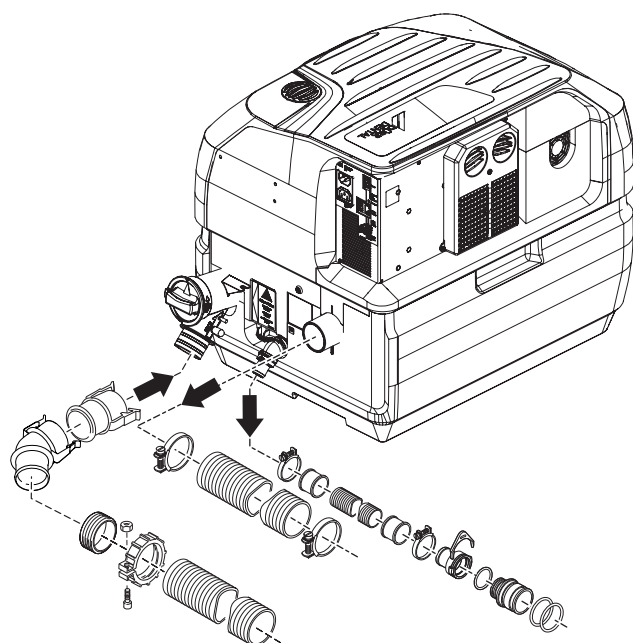

2

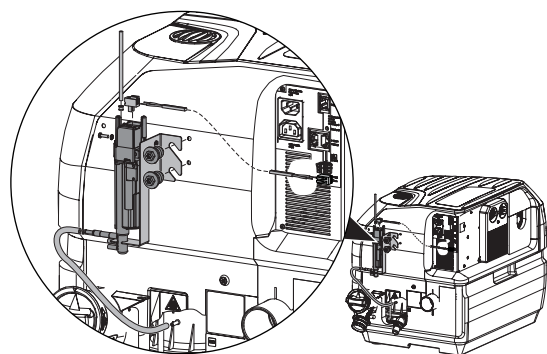

3

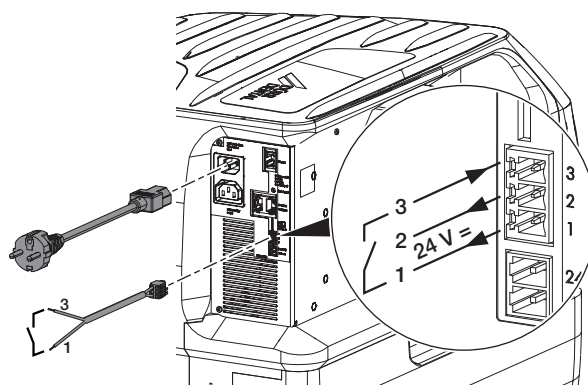


## Usage

### LEDs

- |                          |  |  |
|--------------------------|--|--|
| Ethernet link            |  | The machine is connected to the network            |
| fault                    |  | Fault/failure of a radial blower                   |
|                          |  | Failure of both radial blowers or separation error |
| ready                    |  | Ready for operation                                |
|                          |  | EcoStop  |
| start signal             |  | Manifold signal active, machine running            |
| ⚙ Status LED flashing    |  |  |
| ○ Display is illuminated |  |  |



## Disinfection and cleaning

After each patient: suction 0.5 l of water

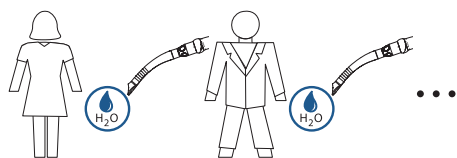

### Daily: disinfection




Daily in the evening after finishing treatment; twice daily with high loads.

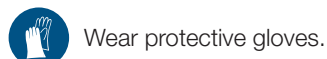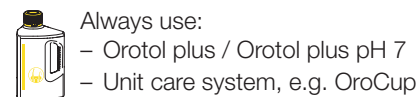

Wear protective gloves.





Wear protective goggles.

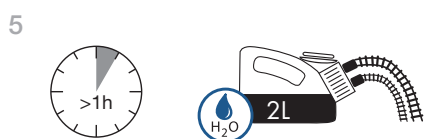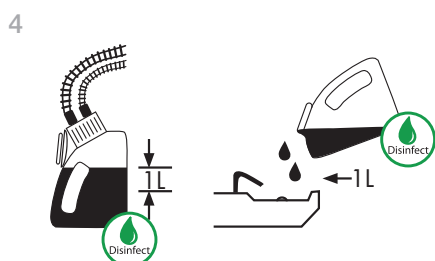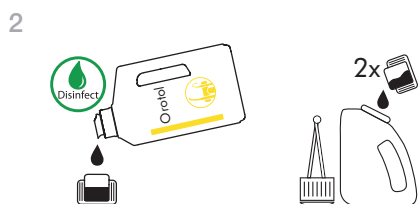

## Cleaning: 2 x or 5 x per week

Min. 2 x per week, contact time 30 - 120 minutes




Alternatively: 5 x per week, contact time 5 - 15 minutes

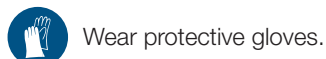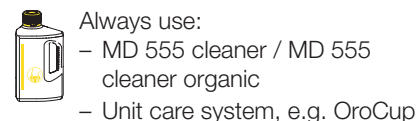

Wear protective gloves.



Wear protective goggles.

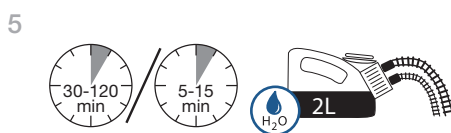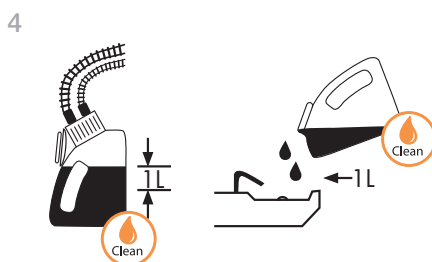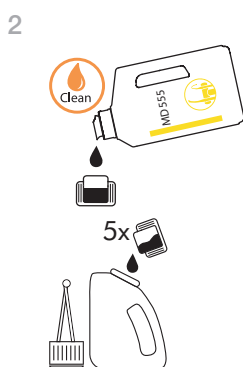

## Clean the filter at the device suction connection.

