



## Tyscor V 1 / V 2 suction unit



Order number: 7177100004




Ambient temperature



Upper limit of humidity




Only qualified specialists or employees trained by Dürr Dental are permitted to install, connect and start using the unit.

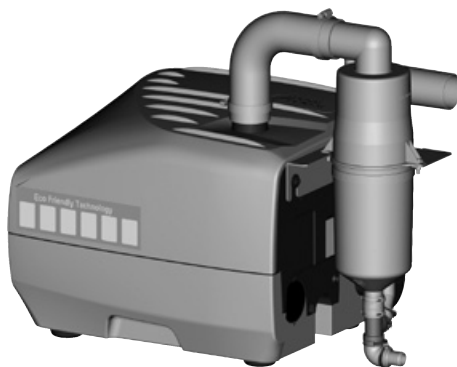


CE 0297

## Assembly

1

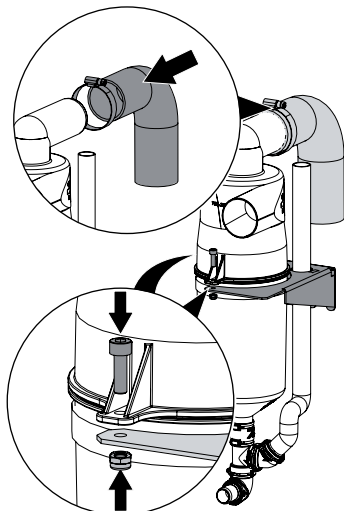

2

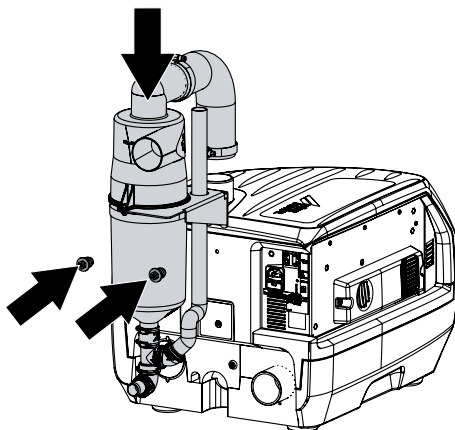

**Hersteller/Manufacturer:**  
DÜRR DENTAL SE  
Höpfheimer Str. 17  
74321 Bietigheim-Bissingen  
Germany


7177100003L01




3 / 4

**DÜRR  
DENTAL** 1905V005

3

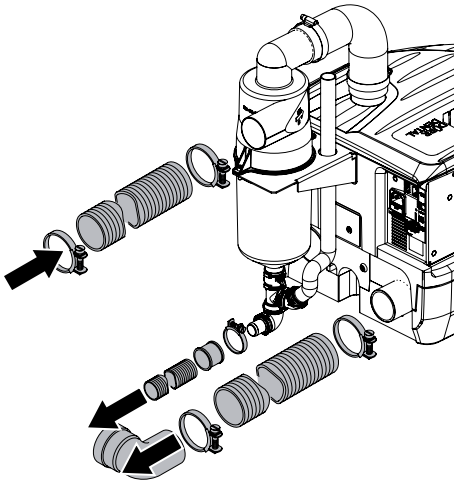


4

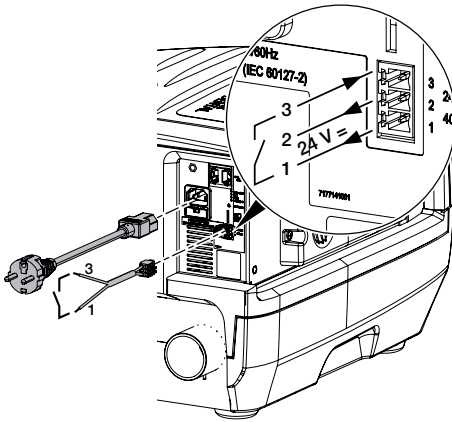

## Usage


1

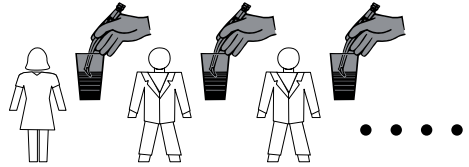


› After treating each patient, aspirate one glass of cold water.

2

**1x / Tag abends**

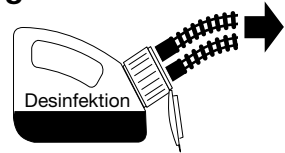

**1x / Woche mittags**

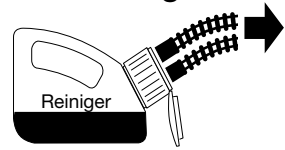

- › Daily, after completing treatment, aspirate a unit of disinfection solution, e.g. Orotol plus.
- › Once a week before the lunch break, aspirate a unit of cleaning agent solution, e.g. MD 555 cleaner. Rinse with water after the reaction time.