



Tyscor VS 1 / VS 2 combination suction unit



Order number: 7186100001



Ambient temperature



Upper limit of humidity




Only qualified specialists or employees trained by Dürr Dental are permitted to install, connect and start using the unit.




CE 0297

Assembly

1

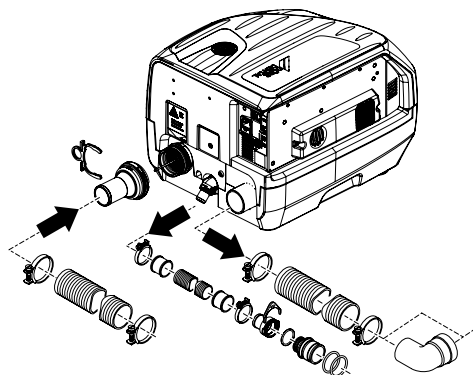

2

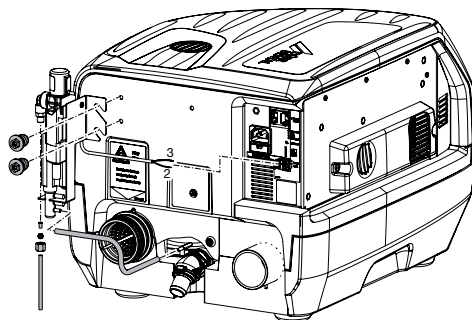

Hersteller/Manufacturer:
DÜRR DENTAL SE
Höpfigheimer Str. 17
74321 Bietigheim-Bissingen
Germany


9000-606-130/01



3 / 4


**DÜRR
DENTAL**

1905V004

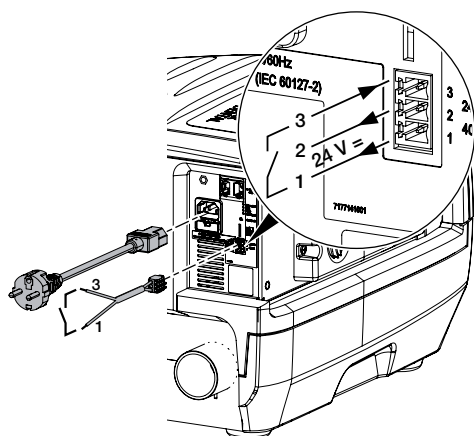

Usage


1

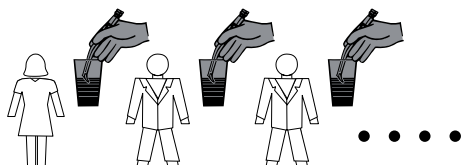


- After treating each patient, aspirate one glass of cold water.

2

1x / Tag abends

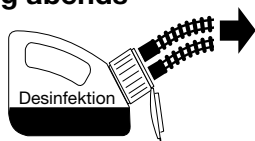


1x / Woche mittags

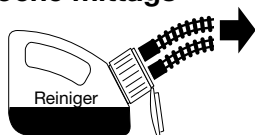


- Daily, after completing treatment, aspirate a unit of disinfection solution, e.g. Orotol plus.
- Once a week before the lunch break, aspirate a unit of cleaning agent solution, e.g. MD 555 cleaner. Rinse with water after the reaction time.

3

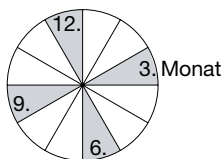

1.

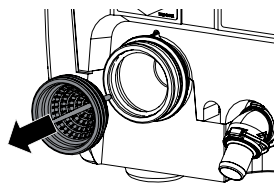

2.



oder **NEU**



- Clean or replace the protective sieve on the suction input every 3 months.