


Installationshinweis CA 4 an bestehende Fernanzeigen

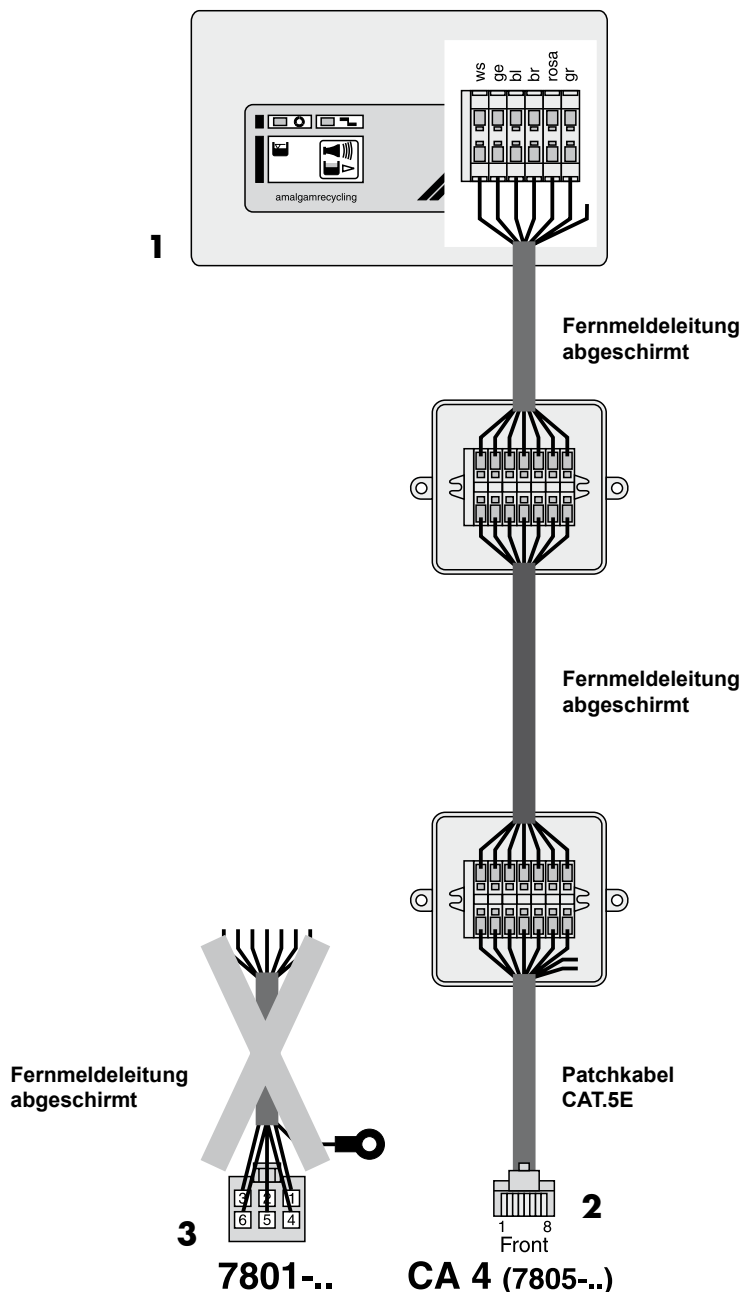

Wird ein Amalgamabscheider Typ 7801 durch einen CA 4 ersetzt so ist das beiliegende Patchkabel entsprechend nachfolgender Tabelle anzuschließen.

Wichtig ist die durchgehende Farbzuordnung, nicht die Klemmenbelegung.

Anschlußbelegung der verschiedenen Stecker, Leitungen und Klemmen

Anzeigemodul (1) Klemmleiste			ALT Amalgamabsch. Stecker 6-pol. (3)		NEU Netzwerk Patchkabel (2)	
Funktion	Klemme	Farbe	Pin	Farbe	Pin	Farbe
LED gelb	1	weiss (ws)	3	weiss	2	orange
Taste	2	gelb (ge)	1	gelb	5	weiss-blau
LED grün	3	blau (bl)	6	blau	1	weiss-orange
Piezosummer	4	braun (br)	2	braun	8	braun
LED rot	5	rosa (rosa)	4	rosa	7	weiss-braun
LED GND	6	grau (gr)	5	grau	4	blau
-	-	-	-	Schirm	-	Schirm
-	-	-	-	-	3	weiss-grün
-	-	-	-	-	6	grün

Dürr Dental SE

Höpfheimer Str. 17 • 74321 Bietigheim-Bissingen/Germany

Tel.: +49 (0) 7142 / 7050 • Fax: +49 (0) 7142 / 61365

eMail: info@duerrdental.com

Internet: <http://www.duerrdental.com>


9000-606-48/01



2005/08SE



CA 4 Installation instructions for existing remote displays

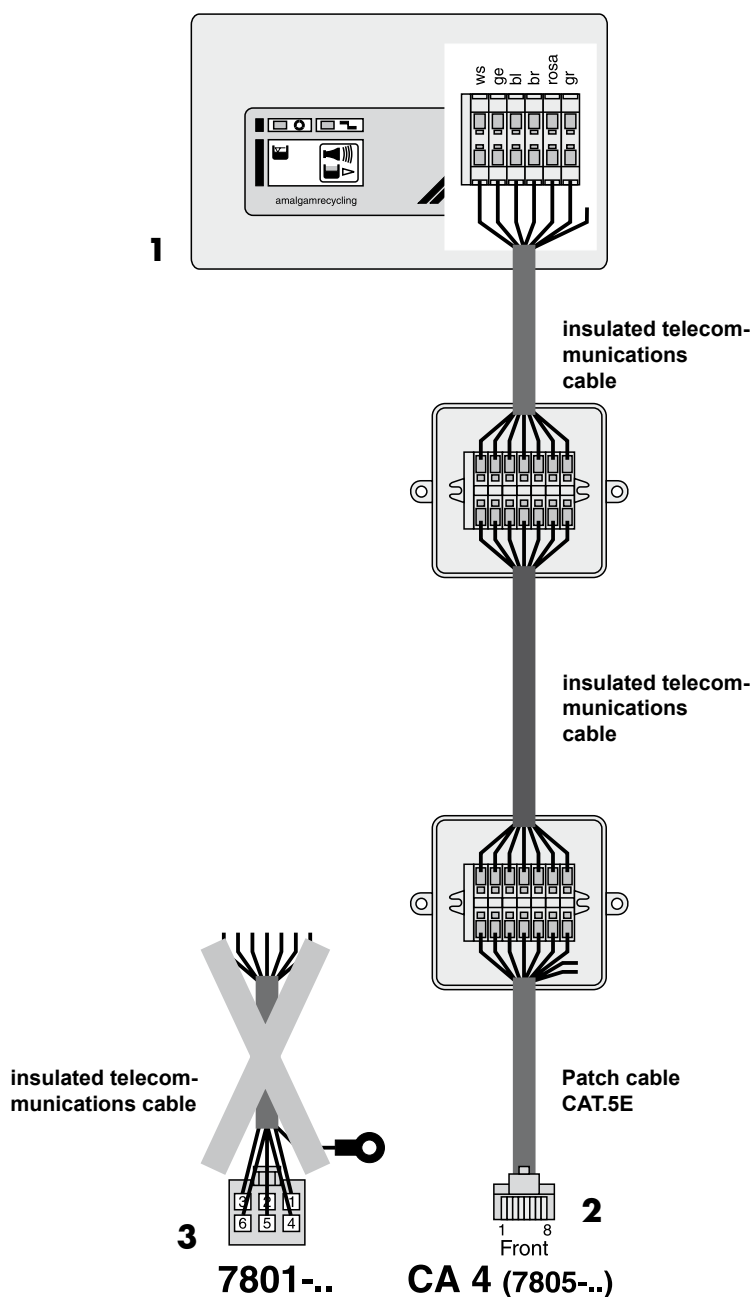

If an amalgam separator of the model 7801 is to be replaced by a CA 4 unit then the Patch cable supplied should be connected according to the following table. It is important to note the wiring colours, the terminal contact order is not critical.

Terminal contacts of the various jumpers, connections and contacts

Display module (1)			OLD Amalgam Separator		NEW Mains	
Terminal strip			Plug 6-wire core (3)		Patch cable (2)	
Function	Contact	Colour	Pin	Colour	Pin	Colour
LED yellow	1	white (ws)	3	white	2	orange
Key	2	yellow (ge)	1	yellow	5	white-blue
LED green	3	blue (bl)	6	blue	1	white-orange
Piezo summer	4	brown (br)	2	brown	8	brown
LED red	5	pink (rosa)	4	pink	7	white-brown
LED GND	6	grey (gr)	5	grey	4	blue
-	-	-	-	shield	-	shield
-	-	-	-	-	3	white-green
-	-	-	-	-	6	green

Dürr Dental SE

Höpfheimer Str. 17 • 74321 Bietigheim-Bissingen/Germany
Tel.: +49 (0) 7142 / 7050 • Fax: +49 (0) 7142 / 61365
eMail: info@duerrdental.com
Internet: <http://www.duerrdental.com>



2005/08SE

