

Duo Tandem / Quattro Tandem



www.duerdental.com

Order number: 4252100034




Ambient temperature




Upper limit of humidity



Only qualified specialists or employees trained by Dürr Dental are permitted to install, connect and start using the unit.

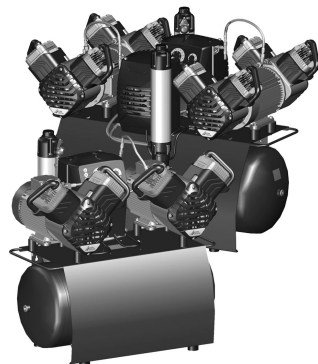

Operating panel

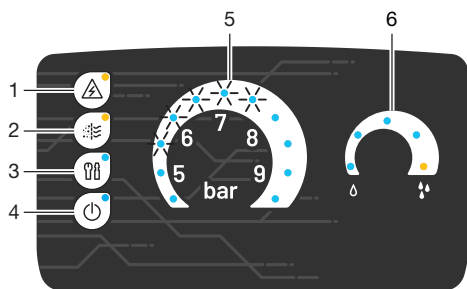

- 1 Fault button with orange LED
- 2 Filter change button with orange LED
- 3 Service key with blue LED
- 4 Standby button with blue LED
- 5 Pressure range display / adjustment
- 6 Pressure dew point display


Switch on the unit.

1




Drain the collecting tray

1

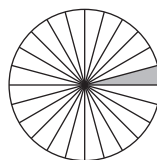

24. Stunde / hour

2

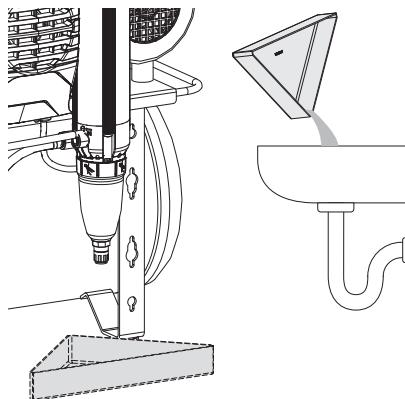

DÜRR DENTAL SE

Höpfigheimer Str. 17
74321 Bietigheim-Bissingen,
Germany

4252100053L40



**DÜRR
DENTAL**

2105V002

Changing the filter


1




2




3

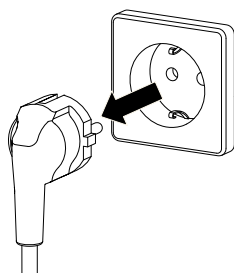


4

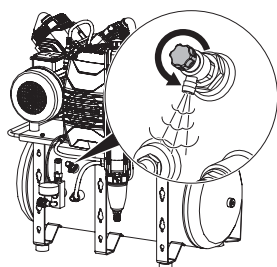


5

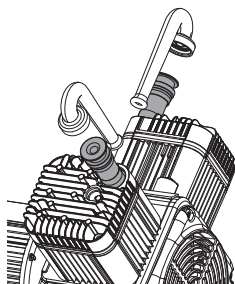


6

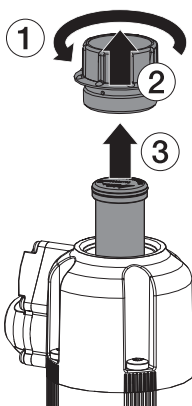

7

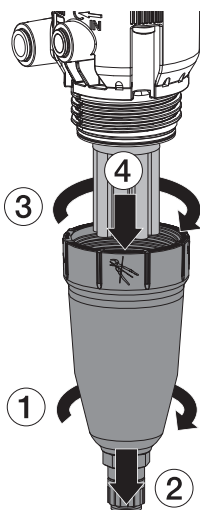


8

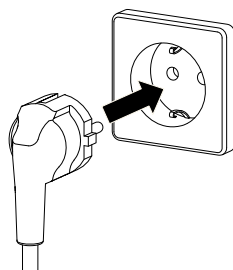

9



10

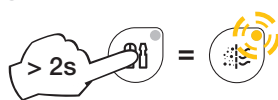

11



12



13

